Heraeus

**Venus<sup>®</sup> White**

Show the world your  
most beautiful smile.

Venus<sup>®</sup>

# Venus<sup>®</sup> White

Celebrate your smile.  
Transform your life.

## Venus® White Whitening Gel Application Instructions

**Important:** Please read before using tooth whitening gel.

Tooth whitening (bleaching) gel will only whiten natural teeth. Results may vary as tooth whiteners do not work on certain stains. It is important to consult with your dentist if you have any concern about stains on your teeth before using this product.

1. Brush and floss teeth. Venus White is most effective on clean teeth.
2. Make sure trays are clean and dry prior to applying Venus White.
3. Remove syringe from case and snap tip cover from the end.
4. To load the tray with Venus White, depress the plunger and fill each tooth in the tray with a pearl of gel. Each tray will typically use 1/2 to 1/3 of a syringe. (Figure 1)

5. Position tray over your teeth and gently press tray to move Venus White gel into place. Wipe excess gel, which seeps over the brim of the tray, off of your gums with a tissue or your finger. (Figure 2)

## Venus® White Whitening Gel Application Instructions (continued)

6. Wear tray for prescribed amount of time depending on your application choice.

**22%—Daytime use:** wear 60 minutes, once a day. Allow a minimum of one hour between each wear time.

**16%—Nighttime use:** wear 8–10 hours, once per day. If sensitivity occurs, discontinue use for 24 hours.

7. After whitening, remove the tray and rinse your teeth. Brush away any remaining Venus White gel.

8. Clean tooth trays with a toothbrush and cold water. Store trays in the tray holder and gel in a cool dry place away from heat and direct sunlight.

Venus White SHOULD BE REFRIGERATED FOR LONG TERM STORAGE. Shelf life is 1 year unrefrigerated and 2 years refrigerated. Use only in accordance with these instructions.

**Warnings:** Keep product away from children. **DO NOT** use if pregnant or breastfeeding. **DO NOT** use tobacco products or eat while wearing bleaching trays.

1. If irritation (such as redness, swelling, soreness) of the gums or the mouth occurs, discontinue use and consult a dentist. Products containing peroxides are not recommended for use by children under 12 years of age.
2. Use for periods of longer than 14 days is to be only under the supervision of a dentist.
3. Avoid swallowing Venus White gel or part thereof.
4. Avoid contact of the product with the eye.
5. Avoid direct contact of the active surface of Venus White with the gums and/or salivary flow.

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**Heraeus Kulzer, LLC**